Packing RBCs and Whole Blood Using the E-54ARC 1-12 RBC Units, or 1-5 WB Units Volume: not less than 350 mL or greater than 4200 mL

Step	Action	Picture	Materials
1	Start Here!		E-54 insulated shipping container
2	Place 1 sheet of absorbent material (folded to fit) on the bottom of the shipping container.	\Diamond	1 sheet absorbent material
3	Place bubble wrap on all sides of the container wall.		bubble wrap
4	When packing 1 - 5 units, place one 48 oz. TSP (Temperature Stabilizing Pack) preconditioned at 1-6 C in the middle of the bottom of the shipping container.		1 TSP (48 oz.)
5	Place bubble wrap on sides of the TSP; ensure that the bottom is covered.		bubble wrap
6	Place the plastic liner inside the container. Place 1 sheet of absorbent material (folded to fit) inside the plastic liner.		1 plastic liner and 1 sheet of absorbent material